Van der Bend  
Medical Supplies

## Instructions for use

# Patch Test Chambers

for contact-allergy testing

Always refer to the instructions and recommendations of the haptens-manufacturer regarding indications and contraindications prior to using this product and, if available, follow regional guidelines for Patch Testing.

### Intended Use:

Van der Bend Patch Test Chambers are designed as an appliance to bring allergens or allergen mixtures into contact with the skin surface. The product is intended for professional use by- or under the supervision of a doctor for:

- Contact allergic research (type IV) (Patients with suspected contact allergy).
- Experimental skin tests and allergy protocols (Clinical examination).

Van der Bend B.V.  
Kloosterweg 34  
3232 LC BRIELLE (NL)  
+31(0)181 418055  
info@vanderbend.nl  
www.vanderbendchambers.nl

### General preparations:

- The preferred location for applying the tapes is the back; alternatively, the arm can be chosen. Before applying the tapes, the patient's skin should be clean, dry and free of any grease.
- Patients with a hairy back should shave it 48 hours prior to treatment.
- Exposure of the test surface to UV light should be avoided  $\pm$  2 weeks prior to the treatment.
- Use of local corticosteroids should be avoided  $\pm$  1 week prior to the treatment.
- No skin-care products such as oils, ointments, lotions, creams, powders (or other products that may affect the condition of the test surface) may be used for a number of days prior to the treatment.
- The patient should avoid showering and strenuous exercise throughout the test phase to prevent the tapes falling off or shifting.
- Scratching due to itching should be avoided during the test phase to avoid influencing the test results.

### Contraindications:

Do not apply to: non-intact or damaged skin. This is to prevent false positive results and by phenomena such as 'angry back' or 'exited skin syndrome'.

### Benefits and general performance properties:

• Compact design • High-quality medically tested materials • Filter papers are uniquely fixed in the chamber without glue, so that – after removing the patch – they are not left on the skin • Quick and easy application • Pre-prepared cover and, if desired, cover by means of a special blister • Tape can be split into 2 strips of 5 chambers by means of a perforation line • Notches for marking with skin marker • Clear indication for bottom and top, left and right • Optimal occlusion • Square shape facilitates the differentiation between an allergic and toxic reaction • Good occlusion • Visible imprint of the chamber on the skin.

### Packaging:

Van der Bend Patch Test Chambers are supplied in cardboard boxes containing 50 units per box (50x10 chambers)

### Please note:

Any serious occurrence related to this device has to be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established. This product has been designed, produced and tested for single application. If the intended use is deviated from and this SUD is reused, unexpected and unpredictable problems may arise that affect the safety, effectiveness or performance of the SUD.

## Instructions for use

### Patch Test Chambers for contact-allergy testing

1. Remove the plaster from the box as shown in the illustration on the inside of the lid. The patient's skin should be thoroughly clean and dry before applying the plaster. Ensure that the plaster is at room temperature before use.
2. Write the numbers of the allergens that will be introduced on the plaster.
3. Introduce the test substance (maximum 25  $\mu$ l) into the chambers without removing the liner from the plaster. You can use the perforation line to split the plaster into 2 strips of 5 chambers.
4. a) Then cover the plaster with the chambers using the special blister and keep it in the refrigerator for use at a later time.

or:

- b) Remove the liner from the "removal point liner" and carefully apply the plaster to the (slightly bent forward) back.

**Note: liquids should only be introduced immediately prior to the plaster being applied.**

5. Press the plaster down firmly between and around the chambers for optimal adhesion.
6. Mark the notches at the top and bottom using a hypo-allergenic skin marker. These notches correspond to the openings in the special read-off template.

The patient should avoid showering and vigorous exertion during the test phase to prevent the plaster becoming loose.

After 48 hours mark the notches once again using a hypo-allergenic skin marker and remove the plaster in the usual way.

If required, use the optionally available read-off template to read the result.

Van der Bend Patch Test Chambers should be stored at room temperature (not above 23°C) and must not be exposed to direct sunlight for long periods.

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